




- LENS MARKINGS**
- ENGRAVED ADD: TEMPORAL SIDE
  - ENGRAVED DIAMONDS: ◇
  - ENGRAVED LOGO: NASAL SIDE
  - HARD RESIN, GLASS, 1.5 TRANSITIONS\*
  - AND TRANSITIONS XTRACTIVE: III
  - THIN & LITE® 1.60 HIGH INDEX: III6
  - POLYCARBONATE: AP
  - ORMEX®: AX
  - ORMEX TRANSITIONS: AX

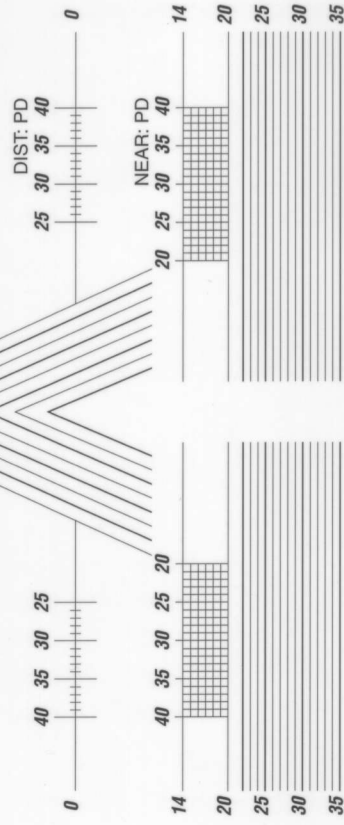


**INSTRUCTIONS**

**CENTERING.** After marking/dotting monocular PD and height on frame demo lenses, center frame on triangle with the dots superimposed on "zero" line. Read R & L Mono PD. Read R & L Mono Height, using lowest point of lower bevel.

**SIZING.** Superimpose dot on fitting cross in lens diagram. The frame must fall within the circle of diameter and material required.


**VERIFICATION.** Upon receipt of glasses from the lab, check markings using the centering instructions to insure proper measurements. If lenses are not marked, they must be remarked before verification.

CENTERED 78 AND 80mm

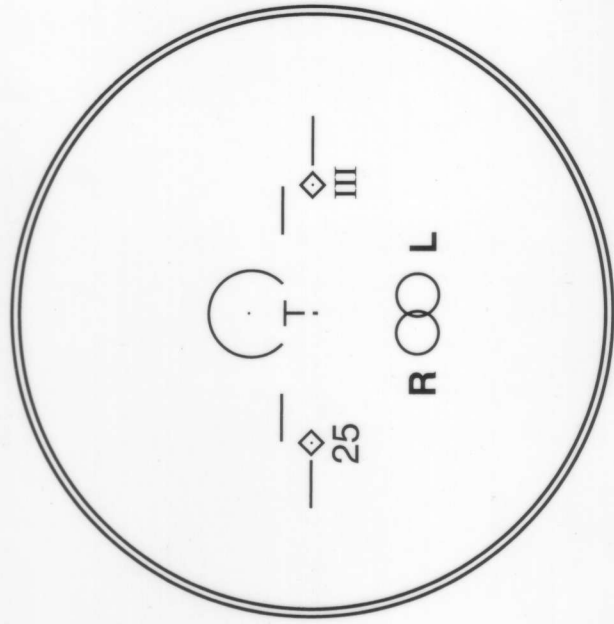

**DECENTERED**

**80 EFFECTIVE** — ORMEX TRANSITIONS HARD RESIN THIN & LITE 1.60 HIGH INDEX POLYCARBONATE ORMEX 1.55 HIGH INDEX **78 EFFECTIVE** — 1.5 TRANSITIONS TRANSITIONS XTRACTIVE **70 EFFECTIVE** — CLEAR GLASS PHOTOGRAY EXTRA®

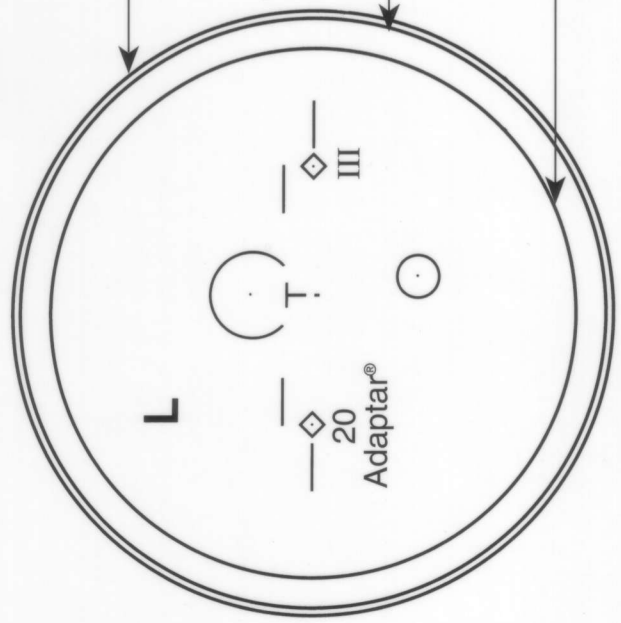

Photogray Extra is a registered trademark of Corning Incorporated, Corning, N.Y.  
 Transitions is a registered trademark of Transitions Optical, Inc.  
 Adaptar No-Line Lens and Design is a registered trademark of Essilor of America, Inc.  
 Ormex is a registered trademark of Essilor International.  
 Thin & Lite is a registered trademark of Essilor of America, Inc.