

Essilor Natural

LENS MARKINGS

ENGRAVED ADD: TEMPORAL SIDE

ENGRAVED CIRCLES: ○

ENGRAVED LOGO: NASAL SIDE

1.50 INDEX PLASTIC, GLASS, 1.50 TRANSITIONS* 2


THIN & LITE® 1.60 HIGH INDEX; 26

ORMEX® TRANSITIONS; 256

AIRWEAR®; 2P

AIRWEAR® TRANSITIONS*

ORMEX; 2X

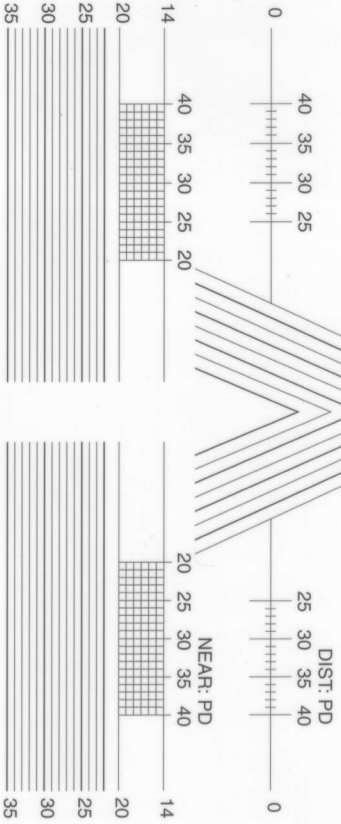

INSTRUCTIONS

CENTERING:

- Mark/dot monocular PD & height on frame demo lenses
- Center frame on triangle with dots superimposed on "O" line
- Read R & L mono PD
- Read R & L mono height using lowest point of lower bevel

SIZING:

Superimpose dot on fitting cross in lens diagram. The frame must fall within the diameter of the circle for the material required.

VERIFICATION:

Upon receipt of glasses from the lab, check markings using the centering instructions to ensure proper measurements. If lenses are not marked, they must be remarked before verification.

TECHNICAL ASSISTANCE (800) 822-5367



©2004 Essilor of America, Inc. Essilor, Airwear and Ormex are registered trademarks of Essilor International, S.A. PhotoGray Extra is a registered trademark of Corning Incorporated, Corning, N.Y. Transitions is a registered trademark of Transitions Optical, Inc. Essilor Natural is a registered trademark of Essilor International, S.A. Thin & Lite is a registered trademark of Essilor of America, Inc. 08/04CP

DECENTERED

80

EFFECTIVE

85

AIRWEAR®

THIN & LITE® 1.60 HIGH INDEX

ORMEX® 1.56 MID INDEX

1.50 INDEX PLASTIC

77

EFFECTIVE

82

ORMEX® TRANSITIONS®

1.50 TRANSITIONS®

AIRWEAR® TRANSITIONS®



70

EFFECTIVE

75

1.60 CLEAR GLASS

& PHOTOGRAY EXTRA®

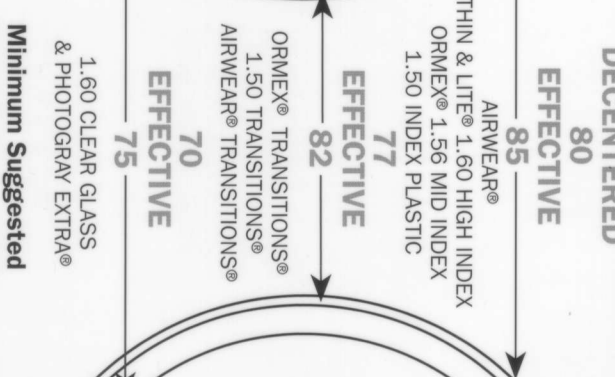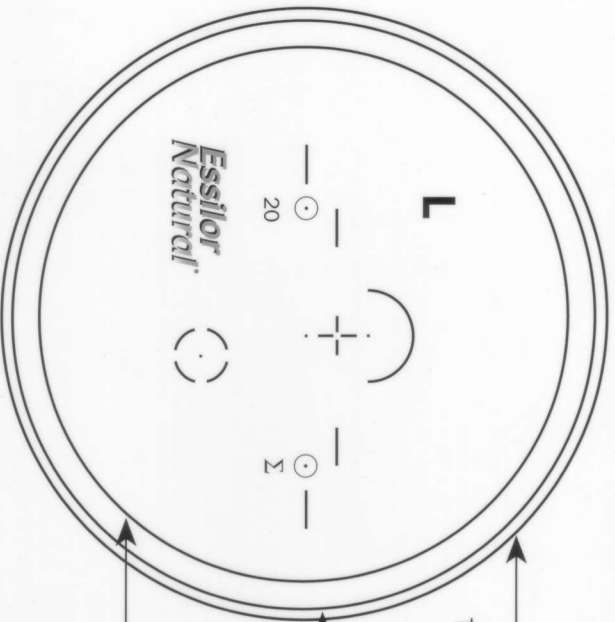

Minimum Suggested
Fitting Height: 18MM