



Sizing

Superimpose dot on fitting cross in lens diagram. The frame must fall within the circle of diameter and material required.

80 HARD RESIN
70/75 GLASS

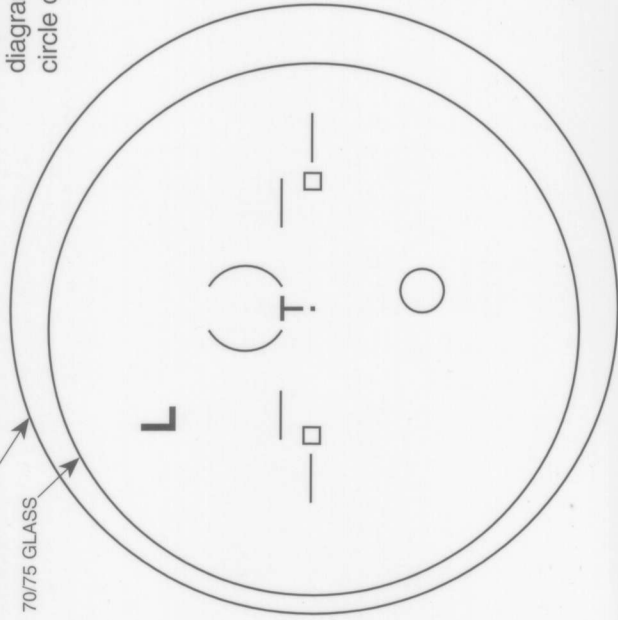

SIZING CHART

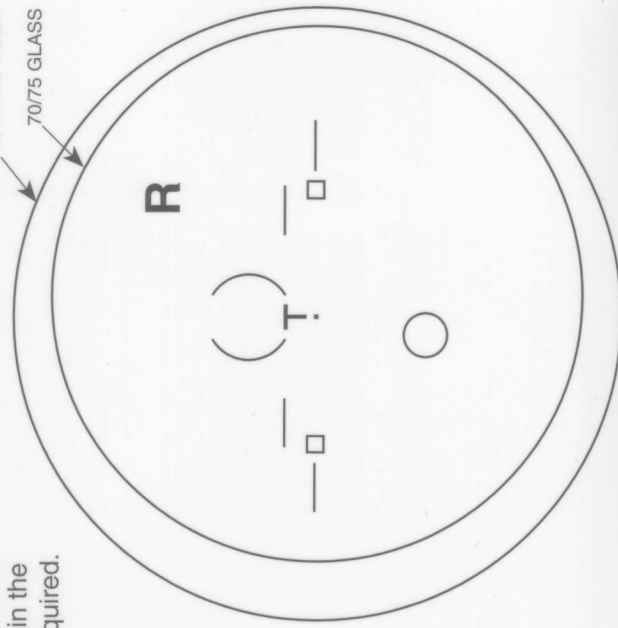

CENTERING CHART

Centering

After marking patients monocular PD and height on frame with Essilor Tape or lenses place frame exactly on center of triangle with the dots superimposed on "O" line. Read R & L Mono PD. Read R & L Mono Ht. using lowest point of lower level.

Verification

Upon receipt of glasses from the lab, check markings using the centering instructions to insure proper measurements. If lenses are not marked, they must be remarked before verification.

